



**K-FILES**



**CORONAL**



**MEDIAN**



**K-FILES**



**APICAL 1**



**APICAL 2**

- Exploration
- Exploring
- Untersuchung

- Évasement coronaire
- Coronal flaring
- Kronenerweiterung

- Élargissement de la partie médiane
- Enlarging the middle part
- Erweiterung des mittleren Abschnitts

- Longueur de travail
- Working length
- Optimale Kanallänge

- Préparation apicale
- Apical preparation
- Vorbereitung des apikalen Abschnitts

- Finition avant obturation
- Finishing before filling
- Reinigung vor dem Füllen

Vitesse : 300 à 400 tr/min – Torque : 2 N/cm

Speed: 300 to 400 rpm – Torque: 2 N/cm

Geschwindigkeit: 300 bis 400 U/Min – Drehmoment: 2 N/cm



- Évasement et désobturation
- Flaring and removal of filling material
- Erweiterung und Entfernen der Füllung

- Évasement de la partie médiane
- Flaring the middle part
- Erweiterung des mittleren Abschnitts

- Désobturation et préparation apicale
- Removal of filling material and apical preparation
- Entfernen der Füllung und Vorbereitung des apikalen Abschnitts

- Évasement et finition
- Flaring and finishing
- Erweiterung und Reinigung

**Vitesse :** 400 à 600 tr/min – **Torque :** 2 N/cm

**Speed:** 400 to 600 rpm – **Torque:** 2 N/cm

**Geschwindigkeit:** 400 à 600 U/Min – **Drehmoment:** 2 N/cm

Marquage CE 0425 / Organisme certificateur : ICIM S.p.A. Fabriqué par CFPM.  
2 bis, chemin du Loup, 93290 Tremblay-en-France – FRANCE Tel : +33 (0)1 45 91 31 82