

CMA Gold, nickel-titanium instruments with **heat treatment**

Endodontic sequence of canal instruments, for canal shaping and retreatment, in continuous rotation mechanized endodontics.

The heat treatment on NiTi alloy provide to CMA Gold:


- Better resistance to fatigue
- More flexibility
- More résistance to fracture

One simple and unique sequence:

- Same instruments for root canal shaping and retreatment
- Same sequence for all canals: simple, curved and narrows from monoradicated teeth and molars

Safe instruments:

- The signs of wear of the file are visible to the naked eye preventing the risk of fracture

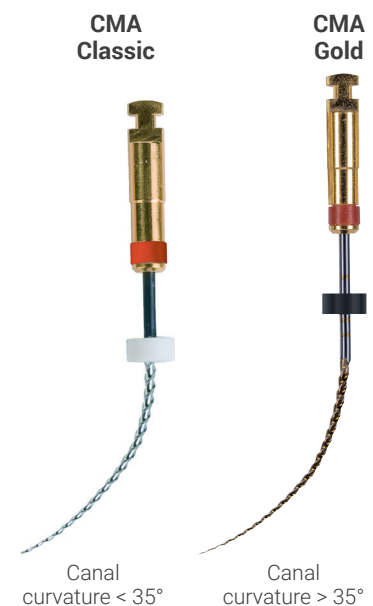

OPERATIVE PROTOCOL DURING TREATMENT

Exploring





➤ Coronal flaring

➤ Enlarging the middle part

➤ Working length

➤ Apical preparation

➤ Finishing before obturation

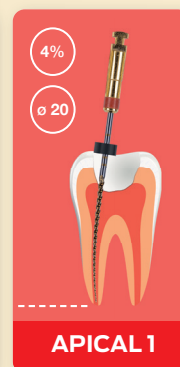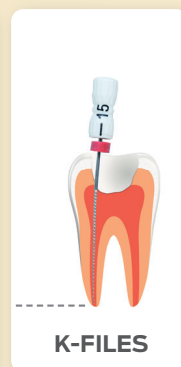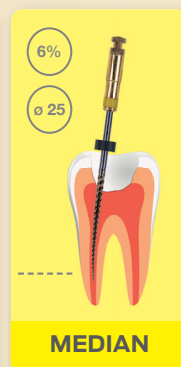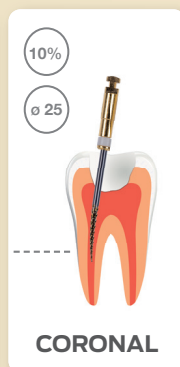

Speed:
300 to 400 rpm
Torque:
2 N/cm

OPERATIVE PROTOCOL DURING RETREATMENT

Flaring and removal of filling material






Enlarging the middle part



Removal of filling material and apical preparation



Flaring and finishing

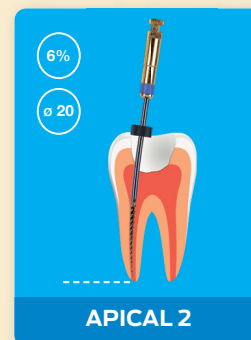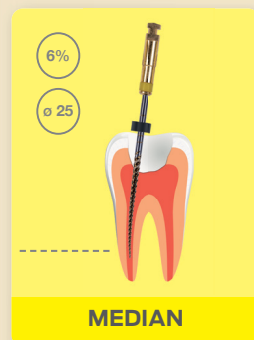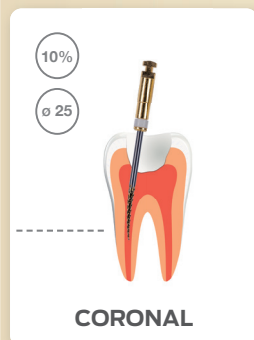

Speed:
400 to 600 rpm
Torque:
2 N/cm

Assorted pack

Contents



CMA GOLD - START KIT A
Length 25 mm

- 4 NiTi GOLD instruments:
1 **CORONAL** 17 mm, 1 **MEDIAN** 25 mm,
1 **APICAL 1** 25 mm, 1 **APICAL 2** 25 mm.
- 2 manual steel files 25 mm No. 10 and No. 15.



CMA GOLD - START KIT B
Length 21 mm

- 4 NiTi GOLD instruments:
1 **CORONAL** 17 mm, 1 **MEDIAN** 21 mm,
1 **APICAL 1** 21 mm, 1 **APICAL 2** 21 mm.
- 2 manual steel files 21 mm No. 10 and No. 15.



CMA GOLD - START KIT C
Length 29 mm

- 4 NiTi GOLD instruments:
1 **CORONAL** 17 mm, 1 **MEDIAN** 29 mm,
1 **APICAL 1** 29 mm, 1 **APICAL 2** 29 mm.
- 2 manual steel files 29 mm No. 10 and No. 15.


Refill of 6 NiTi GOLD rotary instruments

Taper

Ø Tips

Ring

Total length

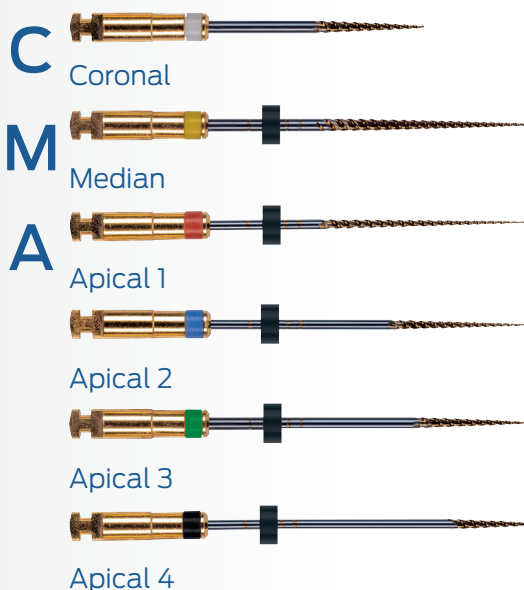

25

White

17 mm



25

Yellow

21 mm
25 mm



20

Red

21 mm
25 mm
29 mm



20

Blue

21 mm
25 mm
29 mm



30

Green

25 mm



40

Black

25 mm

CMA GOLD is manufactured in accordance with European 93/42/EEC and 2007/47/ EC standards.
Class IIa medical devices / CE 0425 marking / Certifying body: ICIM S.p.A.
Follow the manufacturer recommendations for use. Images are for illustrative purposes only.
Manufactured by CFPM. 2 bis, chemin du Loup.
93290 Tremblay-en-France – FRANCE Tel : +33 (0)1 45 91 31 82

www.endo-cma.com